

## 50C1 Necky Optima 50C20 Necky Color 50C30 Necky Color Forte

<b>DE</b> Gebrauchsanweisung .....	3
<b>EN</b> Instructions for use .....	7
<b>FR</b> Instructions d'utilisation .....	10
<b>IT</b> Istruzioni per l'uso .....	14
<b>ES</b> Instrucciones de uso .....	18
<b>PT</b> Manual de utilização .....	22
<b>NL</b> Gebruiksaanwijzing .....	26
<b>SV</b> Bruksanvisning .....	30
<b>DA</b> Brugsanvisning .....	34
<b>NO</b> Bruksanvisning .....	38
<b>FI</b> Käyttöohje .....	41
<b>PL</b> Instrukcja użytkowania .....	45
<b>HU</b> Használati utasítás .....	49
<b>CS</b> Návod k použití .....	53
<b>HR</b> Upute za uporabu .....	56
<b>BG</b> Инструкция за употреба .....	60
<b>EL</b> Οδηγίες χρήσης .....	64
<b>RU</b> Руководство по применению .....	68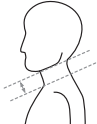

	Nackenhöhe / Neck height	
	cm	inch
	8	3.2
	10	3.9
	12	4.7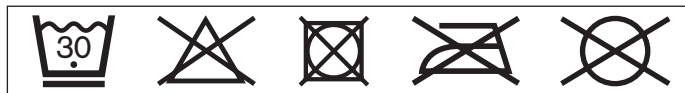

<b>Material</b>	PA, PES, PUR, PCM
-----------------	-------------------

## INFORMATION

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## INFORMATION

Last update: 2018-03-28

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

These instructions for use provide important information on the application of the 50C1 Necky Optima, 50C20 Necky Color and 50C30 Necky Color Forte cervical orthoses.

## 2 Intended use

### 2.1 Indications for use

The orthosis is intended **exclusively** for orthotic fittings of the cervical spine and **exclusively** for contact with intact skin.

The orthosis must be used in accordance with the indications.

### 2.2 Indications

- Cervical syndrome
- Hypermobility of the cervical spine
- Rheumatic complaints of the cervical spine
- Excessive strain on the neck muscles
- Whiplash trauma of the cervical spine
- Degenerative changes of the cervical spine

Indications must be determined by the physician.

### 2.3 Contraindications

#### 2.3.1 Absolute Contraindications

Not known.

#### 2.3.2 Relative Contraindications

Consult the physician if any of the following occur: skin diseases/injuries, inflammation, raised scars with swelling, reddening, and warming in the area where the orthosis is fitted, tumours, lymphatic congestion and dysaesthesia in the cervical spine.

### 2.4 Effects

The orthosis supports and warms the cervical spine.

## 3 Safety

### 3.1 Explanation of warning symbols

**CAUTION** Warning regarding possible risks of accident or injury.

**NOTICE** Warning regarding possible technical damage.

### 3.2 General safety instructions

**CAUTION**

**Contact with heat, embers or fire**

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

**CAUTION**

**Reuse on other persons and improper cleaning**

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

**NOTICE**

**Improper use and changes**

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

**NOTICE**

**Contact with oils, salves, lotions or other products that contain oils or acids**

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

## 4 Handling

### INFORMATION

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

### 4.1 Size Selection

- 1) Measure the neck size.
- 2) Measure the neck height.
- 3) Determine the orthosis size (see size chart).

### 4.2 Application

#### CAUTION

##### **Incorrect or excessively tight application**

Risk of local pressure and constriction of blood vessels and nerves due to improper or excessively tight application

- ▶ Ensure that the product is applied properly and fits correctly.

#### NOTICE

##### **Use of a worn or damaged product**

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

- > Open all hook-and-loop closures on the orthosis.
- 1) Place the orthosis around the neck (see fig. 1).
- 2) Close the orthosis at the back (see fig. 2).
- When looking straight ahead, the orthosis rests on the clavicle and supports the chin (see fig. 3).

## 4.3 Cleaning

### NOTICE

#### Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- Only clean the product with the approved cleaning agents.

- 1) Fasten all hook-and-loop closures.
- 2) Hand wash the orthosis in warm water at **30 °C** with standard mild detergent.
- 3) Rinse well.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

## 5 Disposal

Dispose of the product in accordance with national regulations.

## 6 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 6.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 6.2 CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

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