
## 50S19, 50S23, 50S191

<b>DE</b> Gebrauchsanweisung .....	4
<b>EN</b> Instructions for use .....	8
<b>FR</b> Instructions d'utilisation .....	11
<b>IT</b> Istruzioni per l'uso .....	15
<b>ES</b> Instrucciones de uso .....	19
<b>PT</b> Manual de utilização .....	23
<b>NL</b> Gebruiksaanwijzing .....	27
<b>SV</b> Bruksanvisning .....	31
<b>DA</b> Brugsanvisning .....	35
<b>NO</b> Bruksanvisning .....	39
<b>FI</b> Käyttöohje .....	43
<b>PL</b> Instrukcja użytkowania .....	46
<b>HU</b> Használati utasítás .....	50
<b>CS</b> Návod k použití .....	54
<b>RO</b> Instrucțiuni de utilizare .....	58
<b>HR</b> Upute za uporabu .....	62
<b>SL</b> Navodila za uporabo .....	66
<b>BG</b> Инструкция за употреба .....	69
<b>TR</b> Kullanma talimatı .....	73
<b>EL</b> Οδηγίες χρήσης .....	77
<b>RU</b> Руководство по применению .....	81
<b>ZH</b> 使用说明书 .....	86

Infinity	Art-Nr.	Max. Körpergewicht / User weight limit	Schuhgröße / Shoe Size		
			EU	US	
				Männer / Men's	Frauen / Women's
<b>Air Walker high</b>	50S19=S	<b>200 kg 440 lbs</b>	36-38	4.5-7	6-8
	50S19=M		39-42	7.5-10.5	8.5-11.5
	50S19=L		43-46	10.5-12.5	11.5-13.5
	50S19=XL		47+	12.5+	13.5+
<b>Air Walker low</b>	50S23=S	<b>200 kg 440 lbs</b>	36-38	4.5-7	6-8
	50S23=M		39-42	7.5-10.5	8.5-11.5
	50S23=L		43-46	10.5-12.5	11.5-13.5
	50S23=XL		47+	12.5+	13.5+
<b>Achilles Walker</b>	50S191=S	<b>200 kg 440 lbs</b>	36-38	4.5-7	6-8
	50S191=M		39-42	7.5-10.5	8.5-11.5
	50S191=L		43-46	10.5-12.5	11.5-13.5
	50S191=XL		47+	12.5+	13.5+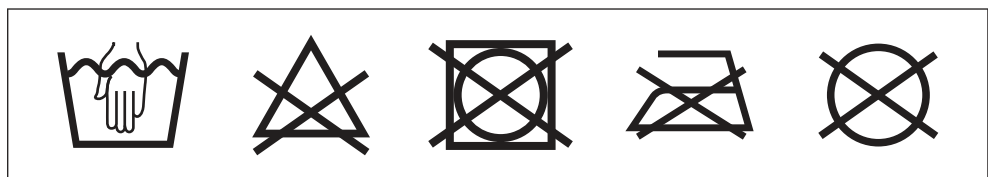

<b>Material</b>	Liner: PA, PU Schaumstoff / PU Foam Walker: PP
-----------------	---

**INFORMATION**

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### INFORMATION

Date of last update: 2020-05-25

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

The instructions for use provide you with important information for adjusting and applying the 50S19 Infinity Air Walker high, 50S23 Infinity Air Walker low and 50S191 Infinity Achilles Walker immobilisation orthoses.

## 2 Intended use

### 2.1 Indications for use

The orthosis is intended **exclusively** for orthotic fittings of the lower limbs and **exclusively** for contact with intact skin.

The orthosis must be used in accordance with the indications.

### 2.2 Indications

#### 50S19 Infinity Air Walker high

- Postoperative, conservative treatment of stable forefoot, metatarsal and/or ankle fractures
- Postoperative, conservative treatment of distal fibula fractures
- Soft tissue injuries of the foot
- Severe ankle sprains

#### 50S23 Infinity Air Walker low

- Postoperative, conservative treatment of stable forefoot, metatarsal and/or ankle fractures
- Soft tissue injuries of the foot
- Exostosis treatment for hallux valgus

#### 50S191 Infinity Achilles Walker

- Postoperative, conservative treatment of Achilles tendon ruptures

Indications must be determined by the physician.

### 2.3 Contraindications

#### 2.3.1 Absolute Contraindications

Not known.

#### 2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent scars that are swollen, reddening and hyperthermia of the fitted limb; pronounced varicose veins, especially with return flow impairment, lymphatic flow disorders, including unclear soft tissue swelling distant to the body area to which the support will be applied; sensory and circulatory disorders in the area of the lower leg and the foot, e.g. in case of diabetic neuropathy.

## 2.4 Effects

The orthosis serves to immobilise the foot/ankle joint area.

## 3 Safety

### 3.1 Explanation of warning symbols

Warning regarding possible risks of accident or injury.

Warning regarding possible technical damage.

### 3.2 General safety instructions

#### Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

#### Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

#### Contact with oils, salves, lotions or other products that contain oils or acids

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

#### Improper use and changes

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

#### Exposure of the product to unsuitable environmental conditions

Damage, brittleness or destruction due to improper handling

- ▶ Avoid storage in condensing ambient humidity.
- ▶ Avoid contact with abrasive substances (e.g. sand, dust).
- ▶ Do not expose the product to temperatures below -10 °C or above +50 °C (e.g. sauna, excessive sunlight, drying on a radiator).

## 4 Handling

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.

- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

#### 4.1 Size Selection

- ▶ Select the orthosis size based on the shoe size (see sizing table).

#### 4.2 Fitting and Application

##### **⚠ CAUTION**

##### **Incorrect or excessively tight application**

Risk of local pressure and constriction of blood vessels and nerves due to improper or excessively tight application

- ▶ Ensure that the product is applied properly and fits correctly.

##### **⚠ CAUTION**

##### **Use with heat-sensitive skin**

Skin irritation due to overheating

- ▶ Do not use the product in case of a known heat allergy.
- ▶ Do not continue to use the product if skin irritation occurs.

##### **NOTICE**

##### **Use of a worn or damaged product**

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

##### **NOTICE**

##### **Incorrect setting**

Damage to the product due to incorrectly assembled or incorrectly adjusted components

- ▶ The product may only be adjusted by qualified personnel.
- ▶ Do not make any improper changes to the settings.

- > Open all hook-and-loop closures on the orthosis.
- > Remove the front shell from the orthosis.
- 1) **Only Achilles Walker:** Position the heel wedge (29S35) in the liner.
- 2) Position the foot in the orthosis (see fig. 1). Check the position of the heel whilst doing this.
- 3) **INFORMATION: Do not change the position of the toe protector.**
- 4) Close all hook-and-loop closures on the liner (see fig. 2).
- 5) Position the front shell so it is centred on the liner (see fig. 3).
- 6) Close all hook-and-loop closures on the Walker one after the other, starting at the top (see fig. 4).

#### Increasing compression

##### **⚠ CAUTION**

##### **Pumping up the air cushions too much**

Risk of local pressure and constriction of adjacent blood vessels and nerves due to excessively tight application

- ▶ Only fill the air cushions with enough air to impede relative movements between the body part and the orthosis.

- 1) To inflate, turn the rotary valve to the right [ Inflate ] (see fig. 5, item 1).
- 2) Push the pump (see fig. 5, item 2) as often as required to achieve the desired compression.
- 3) **Optional:** to deflate, turn the rotary valve to the left [ Deflate ] (see fig. 5, item 1).

## 5 Cleaning

### NOTICE

#### Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

Clean the orthosis regularly:

#### Liner:

- 1) Remove the liner from the orthosis frame.
- 2) Fasten all hook-and-loop closures.
- 3) Hand wash the liner in warm water at 30 °C with standard mild detergent. Rinse thoroughly.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

#### Orthosis frame:

- 1) Wipe with a damp cloth as needed.
- 2) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

## 6 Disposal

Dispose of the product in accordance with national regulations.

## 7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 7.1 Local Legal Information

Legal information that applies **exclusively** to specific countries is written in the official language of the respective country of use after this chapter.

### 7.2 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 7.3 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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