50C20 Necky Color 50C30 Necky Color Forte

DE Gebrauchsanweisung	4
EN Instructions for use	8
FR Instructions d'utilisation	11
IT Istruzioni per l'uso	15
ES Instrucciones de uso	19
PT Manual de utilização	23
NL Gebruiksaanwijzing	27
SV Bruksanvisning	31
DA Brugsanvisning	35
NO Bruksanvisning	39
FI Käyttöohje	42
PL Instrukcja użytkowania	46
HU Használati utasítás	50
CS Návod k použití	54
HR Upute za uporabu	58
BG Инструкция за употреба	62
EL Οδηγίες χρήσης	66
RU Руководство по применению	70
ZH 使用说明书	74

	Artikelnummer Article number	Farbe Colour	Nackenhöhe Neck height	
			cm	inch
	50C20=2-8* 50C30=2-8*	*-0=champagner/ champagne *-1=grau/grey	8	3.2
	50C20=2-10* 50C30=2-10*	*-0=champagner/ champagne *-1=grau/grey	10	3.9
	50C20=2-12* 50C30=2-12*	*-0=champagner/ champagne *-1=grau/grey	12	4.7

Material	
50C20=2	PES, EL, PU foam, PA
50C30=2	PES, EL, PU foam, PA, TPU

INFORMATION

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INFORMATION

Date of last update: 2021-01-26

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide important information on the application of the 50C20=2 Necky Color and 50C30=2 Necky Color Forte cervical orthoses.

2 Intended use

2.1 Indications for use

The orthosis is intended **exclusively** for orthotic fittings of the cervical spine and **exclusively** for contact with intact skin.

The orthosis must be used in accordance with the indications.

2.2 Indications

- Cervical syndrome
- Hypermobility of the cervical spine
- Rheumatic complaints of the cervical spine
- Excessive strain on the neck muscles
- Whiplash injuries of the cervical spine
- Degenerative changes of the cervical spine

Indications must be determined by the physician.

2.3 Contraindications

2.3.1 Absolute Contraindications

Not known.

2.3.2 Relative Contraindications

Consult the physician if any of the following symptoms occur: skin diseases/injuries, inflammation, raised scars with swelling, reddening, and

warming in the area where the orthosis is fitted, tumours, lymphatic congestion and dysaesthesia in the cervical spine.

2.4 Modes of action

The orthosis supports and warms the cervical spine.

3 Safety

3.1 Explanation of warning symbols

CAUTION Warning regarding possible risks of accident or injury.

NOTICE Warning regarding possible technical damage.

3.2 General safety instructions

CAUTION

Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

CAUTION

Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

NOTICE

Contact with oils, salves, lotions or other products that contain oils or acids

Insufficient stabilisation due to loss of material functionality

- ▶ Do not expose the product to oils, salves, lotions or other products that contain oils or acids.

NOTICE

Improper use and changes

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

4 Handling

INFORMATION

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

4.1 Size Selection

The orthosis is intended for a neck circumference of 37 to 45 cm.

- 1) Measure the neck height.
- 2) Determine the orthosis size (see size chart).

4.2 Application

CAUTION

Incorrect or excessively tight application

Risk of local pressure and constriction of blood vessels and nerves due to improper or excessively tight application

- ▶ Ensure that the product is applied properly and fits correctly.

NOTICE

Use of a worn or damaged product

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

- > Open all hook-and-loop closures on the orthosis.
 - 1) Place the orthosis around the neck (see fig. 1).
 - 2) Close the orthosis at the back (see fig. 2).
- When looking straight ahead, the orthosis rests on the clavicle and supports the chin (see fig. 3).

5 Cleaning

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

- 1) Fasten all hook-and-loop closures.
- 2) Hand wash the orthosis in warm water at **30 °C** with standard mild detergent.
- 3) Rinse well.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

6 Disposal

Dispose of the product in accordance with national regulations.

7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

7.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

7.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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